


Oral Rinse Collection Instructions

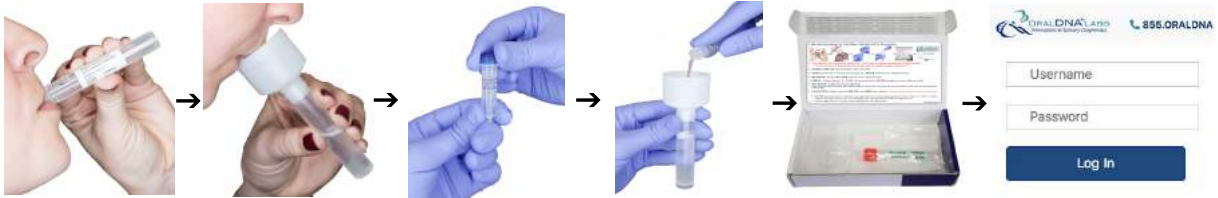

1. **VIGOROUSLY SWISH & GARGLE ALL** 5 mL of saline solution for **AT LEAST 30 SECONDS**
2. **SPIT** saline into funneled collection tube; **POUR** preservative (blue cap) into collection tube
3. **DISCARD** funnel; **SEAL** collection tube with red cap
4. **PRINT** "Patient Name" & "DOB" on barcode label; **AFFIX** lengthwise onto collection tube
5. **PACK** box with bagged collection tube; **REMOVE** adhesive liner; **CLOSE & SEAL** box
6. **LOGIN** to oraldna.com to **CREATE** test order(s); **SHIP** via **US MAIL**

OralDNA® Labs **MUST** receive sample within **21 DAYS** from the date collected.

© OralDNA® Labs 2021, all rights reserved PL-000490, Rev C